INSTRUKTION

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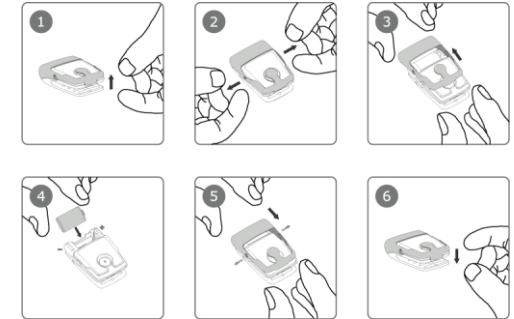
Disclaimer:

This baby monitor is a parenting aid and not a medical device or a substitute for human care-giving. It is intended for use under safe operating conditions and must be used responsibly and in accordance with the detailed instructions appearing on this leaflet. The diagnosis and monitoring of disease must be done by a medical professional. Premature and other high-risk infants and infants with a known predisposition to apnoea should be monitored by a doctor. This product cannot address all respiratory threats and is not designed for use in life support situations. Because the monitor supervises tummy movement and not air flow it is not ideal for detecting obstructive apnoea. Customers use this monitor at their own risk and Infantrust Parenting Solutions (Proprietary) Limited, its staff, agents, contractors, manufacturers, distributors and associates ("INFANTRUST") shall not be held liable for any loss, damage or injury, including consequential loss, damage or injury, suffered by the customer or any other person arising from or in connection with the use or manufacture of the monitor, including loss, damage or injury suffered by the customer or any other person as a result of any defects (whether patent or latent) in respect of the monitor, and all warranties, whether express, implied or tacit, and any other liability from whatsoever cause arising, are hereby expressly excluded and the customer hereby indemnifies and holds harmless INFANTRUST in respect of any such liability.

Technical specifications:

Battery Respisense ER14250M 3.6V Lithium.

Primary alarm Fast warbling tone.

Secondary alarm Slow warbling tone.

Designed and manufactured in South Africa by Infantrust Parenting Solutions (Pty) Ltd., PO Box 3316, Matieland, 7602, South Africa.

ZA2006/01793 and GB2419961 apply. Further patents pending. All rights reserved; Respisense, Infantrust, TummyTickle and the associated logos are Trademarks.

To improve Respisense monitors, the appearance and specifications of the monitor may change without notice.